

VERIFY® Bowie Dick Test Card is the smallest and most simple pre-assembled Bowie Dick test pack available. This Card is the first test that generates no waste at all, because the Card is the test. Upon completion of the Bowie Dick test cycle, you will have immediate validation of the air removal process.



## Verify® Bowie Dick Test Card

### Air Removal Test for Steam Sterilizers

#### EASY TO USE

In an otherwise empty chamber, place the VERIFY Card, with chemical indicator bars facing up, in a test card holder on the lowest shelf over the sterilizer drain. During the vacuum phase of the cycle, air is removed from this uniquely designed product. Following air removal, the chemical bars uniformly darken with exposure to steam. If air is trapped in the card during the exposure phase of the cycle, all the bars will not show even color change. It's that simple.

Colors are an approximation




**PASS**



**TYPICAL FAILURE**

Each VERIFY Bowie Dick Test Card box offers convenient and permanent storage for two months of processed cards. No color fading or darkening after processing gives you reliable and permanent documentation. The most important benefit of the VERIFY Bowie Dick Test Card is that it eliminates all medical waste associated with other pre-assembled tests. All this from a product the size of a credit card.

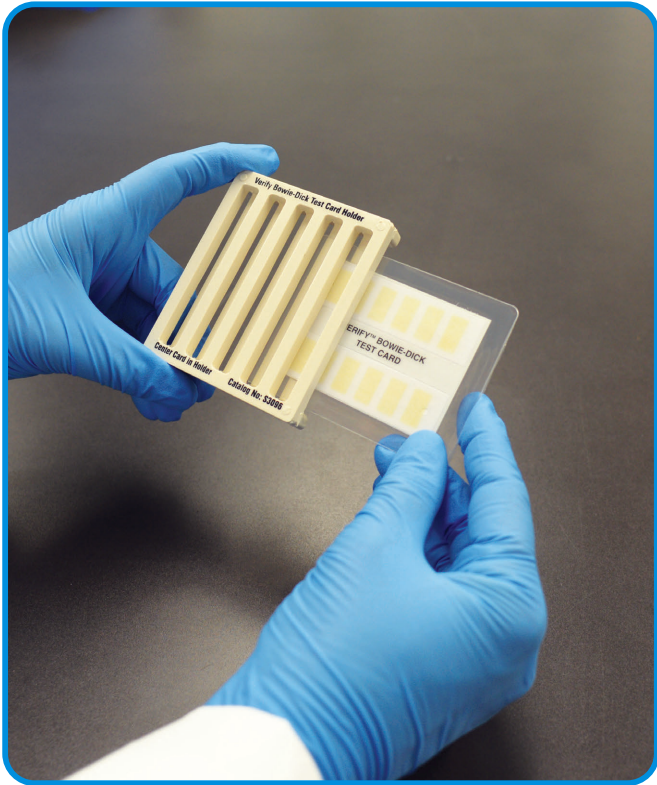

### OPERATING PARAMETERS

- 132°-134°C (270°-274°F) cycle temperature
- 3.5 to 4.0 minutes/cycle based on temperature setting

### CYCLE TYPE

- Pre-vacuum

### STANDARD

- ISO 11140

### SHELF LIFE

- 36 months from time of manufacture

RECOMMENDED FOR DAILY USE

### ORDERING INFORMATION

Product	Description	Packaging
S3095	VERIFY Bowie Dick Test Card	31/Box
S3096	VERIFY Test Card Holder	1/Box